## Contact MCPS Systemwide Safety Programs (301-370-2141) immediately if any problems are found

## **AED Monthly Inspection Form – Keep With AED**

Scan for MCPS AED information and forms



Facility:	AED Serial No. (optional):
AED Location:	

Inspection Date	Inspector Initials	AED and carrying case (and wall cabinet, if applicable) intact?	All electrodes (chest pads) sealed, intact, and not expired?	"OK" symbol displayed or Green Status Light flashing?	All required equipment/forms with AED?**

<sup>\*\*</sup>Required equipment and forms (must be with AED at all times):

- 2 sets of adult chest pads 1 set installed in the AED tightly, 1 back-up set pads must be replaced if package has been opened
- 1 set of infant/child chest pads (except for Lifepak CR2 AEDs, which do not need separate infant/child pads)
- AED Monthly Inspection Form
- Resuscitation kit (2 pairs latex-free disposable gloves, CPR mask, pediatric CPR mask in pouch, shears, razor)